



User Information Guide

Inflatable T-Roll

How to use the Inflatable T-Roll

Before unwrapping the support, please check that the product is the correct size and a good fit. As unwrapped products cannot be returned, credited or exchanged due to health and hygiene reasons. See returns policy enclosed.

The T-roll is used to control position of the body in supine lying. Pillows placed laterally along the length of the trunk lend additional support when necessary. This product is designed for short term use only.

When using the T-Roll always support the feet in order to relieve pressure at the back of the leg.

The Inflatable T-Roll should be inflated to the degree to provide a firm shape allowing approximately 12mm (1/2 inch) of depression under moderate pressure. They may be used with a Soft Knit cover available in 5 colours.

Note

As with all new positioning equipment, it should be gradually introduced and the use carefully monitored. If discomfort is experienced such as may interrupt sleep, discontinue use and consult your therapist.

Cleaning

Superficial dirt on the surface may be removed by wiping with a soft cloth moistened with hand hot water and a neutral detergent. Care should be taken when removing and replacing covers. Abrasive cleaning agents should not be used.

Hygiene and Disinfection

Cleaning and disinfection may be undertaken on the exterior covering. Proprietary disinfectants may be used provided the manufacturers' instructions are followed.

Storage

Store in a cool dry area away from direct sunlight.

